

Aconia 3D Master 16 – 26 Shade Conversion Table

Target Shade	Block Shade	Staining Adjustment
1M1	B1	
1M2	B1	Modify with Artamic Shade B Stain
2L1.5	B1	
2L2.5	B2	
2M1	A1	
2M2	A2	
2M3	B2	Modify with Artamic Shade B Stain
2R1.5	A1	
2R2.5	A3	
3L1.5	C2	Modify with Artamic Shade C Stain
3L2.5	B3	Modify with Artamic Shade B Stain
3M1	D2	
3M2	A3	
3M3	B4	
3R1.5	D2	Modify with Artamic Shade D Stain
3R2.5	A3.5	
4L1.5	C3	
4L2.5	B4	
4M1	D3	
4M2	A4	
4M3	A3.5	
4R1.5	C4	Modify with Artamic Shade A Stain
4R2.5	A4	
5M1	C4	
5M2	Soak white block with 5M2 coloring liquid	
5M3	Soak white block with 5M3 coloring liquid	